

# TIXOS SHORT

## Implants for limited vertical space

Code ITX3705 - ITX5005

The external hex Tixos implant line is completed with the short implants designed to be inserted in particular anatomical conditions, i.e. limited vertical space.

The **external hex TiXos Short implants with 5mm height** has been designed to be inserted in particular situations:


- in cases of **limited bone height**
- to **avoid advanced surgery treatments**, i.e. sinus lift and bone regeneration procedures
- as **therapeutic alternative** to more complex and expensive surgical treatments
- as **time and money saving treatment**

Like the whole Tixos implant line, Tixos Short is produced through **Direct Laser Metal Forming technique (DLMF)**.

The small size of short implants of other brands on the market implies limitations of use. The **innovative features** of the **lasermade Tixos surface - porous and tridimensional** - ensure a grater implant/bone contact, thus compensating for its small size.

The lasermade Tixos surface and the fixture geometry ensure better and **faster osseointegration**, as well as better **implant/bone interface**, better **primary stability** and earlier **secondary stability**.

Tixos Short implants  
5 mm height

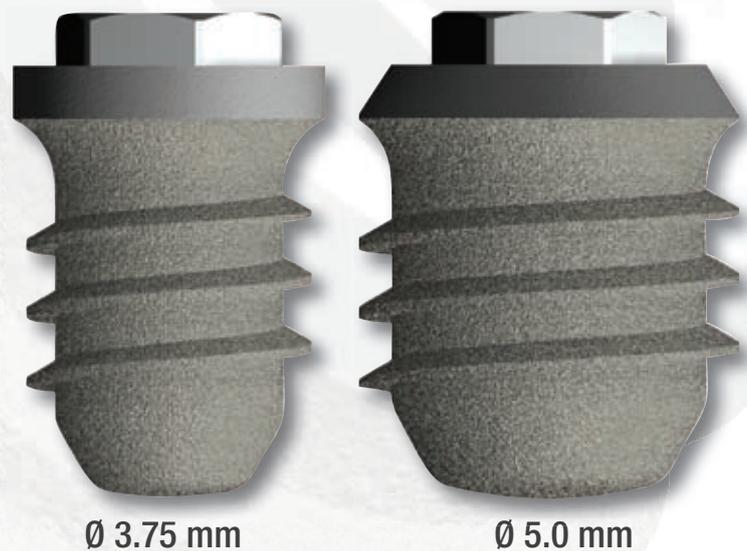

Surgical Box for Tixos Short

# TIXOS SHORT

## Implants for limited vertical space

Code ITX3705 - ITX5005

Tixos Short can be used with Tixos Short surgery kit, complete with the proper 5 mm height Stop kit for Twist drills (cod. STPK4).


The implant package contains the **mount-transfer**, a multi-functional tool that can be used to transport the fixture to the implant site, as impression transfer and temporary abutment.



### TIXOS SHORT EXTERNAL HEX SURGICAL PROTOCOL

First preparation

Ø 3,75 - 5,0

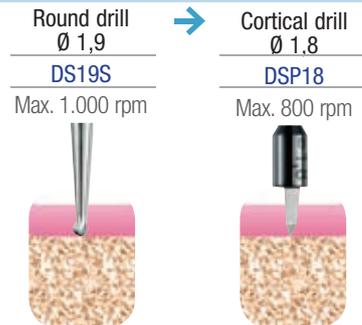


#### Recommended sequence

Ø 3,75

Platform  
Ø 4,1 mm



Code  
**09ITX3705**

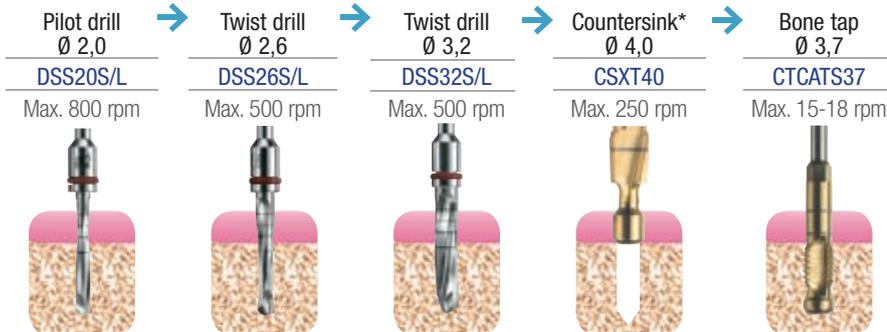


Ø 5,0

Platform  
Ø 4,1 mm



Code  
**09ITX5005**

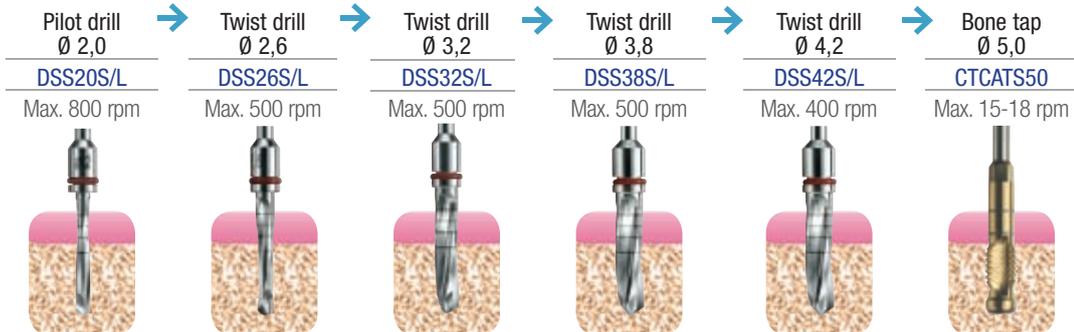

Note: Usually the insertion is operated using a contra-angle.

The technical suggestions reported in this catalogue can under no circumstances substitute the clinical evaluations and therapeutic indications that are of exclusive competence of the Dentist.

LEADER  
ITALIA

**LEADER ITALIA srl** - Cinisello Balsamo MI - ITALY  
ph +39 (0)2 618651 - [www.leaderitalia.it](http://www.leaderitalia.it) - [export@leaderitalia.it](mailto:export@leaderitalia.it)